

Proper Omnilux® lens care instructions

"80% of lens scratches"
occur by cleaning eyewear incorrectly



Always use warm water and mild soap without additives to clean your lenses. Never clean them dry!

You may also use an approved lens cleaning solution provided by your eye-care professional



After washing, dry your lenses with soft lint-free cloth. You may also use a microfiber cloth specially designed for your eyewear.

Never use glass cleaner or other harsh home cleaners including alcohol. Avoid all chemicals like hair spray, perfume or cologne which could cause damage.



- NEVER use facial tissues or paper towel to dry your lenses as they may scratch.
- NEVER leave your eyeglasses in direct sunlight or in a hot car. The heat can damage your lenses.
- NEVER leave your eyeglasses facing downward.

Always keep your eyeglasses in the case when you are not using them!

OmniLux®

NAL® Natural Accommodation Lens

For more information, please contact us at
928-239-9901

Our Eye Doctor:

Dr. Carolyn Martin, OD, MS, FAAO

Our three board certified opticians:

Kayla Ralston, ABO
Optical Manager


Jennifer McIver, ABO
Clinic Manager

Renee Henry, ABO
Optical Consultant

EYE BOUTIQUE OF SEDONA

www.eyeboutiqueofsedona.com

100 Verde Valley School Road
Suite 114
Sedona, AZ 86351

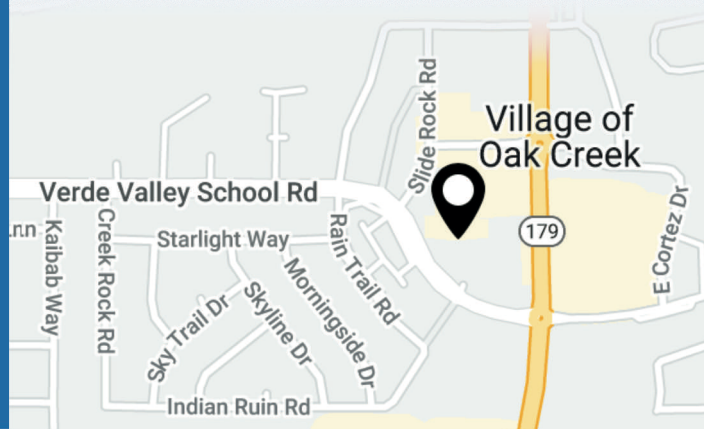

OmniLux®



NAL® Natural Accommodation Lens


What is Omnilux®?

When you receive your brand new Omnilux® lenses you will begin a new journey never experienced before with any other multifocal lens. Omnilux® requires no adaptation period and gives wearers the feeling they are wearing Single Vision lenses with the performance of a multifocal

EYE BOUTIQUE OF SEDONA


NAL[®]

Natural Accommodation Lens

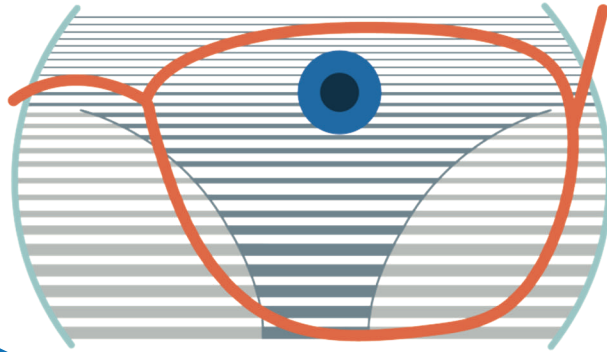


PAL

Progressive Addition Lens

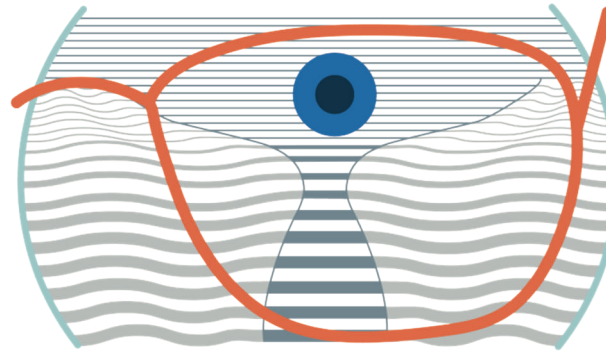

Lifestyle Activities

NAL[®] Focal Length Backbone Deceleration Ramp

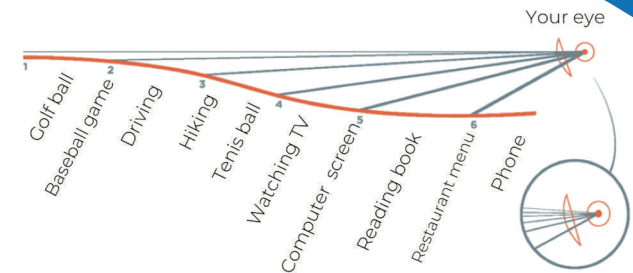

Omnilux[®] B E N E F I T S

- No adaptation period
- No peripheral swim effect
- Provides clear, natural vision in all distances
- No fitting height required

Omnilux[®] lenses are designed **to be worn all day**, and are recommended to all multifocal wearers.

Progressive R E A L I T Y C H E C K

- Adaptation period necessity
- Peripheral swim effect
- Supports clear, limited vision in some distances
- Fitting height is required

Driving glasses Glasses ...
Glasses for work Reading glasses
Glasses ... Glasses for Pilots
Glasses to play Golf Glasses ...

Professional ✓ Leisure ✓ Daily ✓

Natural comfortable vision for all activities

Omnilux[®]
NAL[®] Natural Accommodation Lens

Thanks to Omnilux[®] lenses you can perform all your daily lifestyle activities with natural clear vision.

No adaptation period allows you to enjoy your new glasses immediately.

Omnilux[®] lenses are also suitable for your sunglasses.

Ask your optician for more information